IFU

(Instruction for use)

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Favor contactar seu distribuidor autorizado.

This product has been designed to integrate clinical, mechanical and biological concepts, so, for best results it should only be used with original associated components according to AllPrime Select instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthodontists and Oral Maxillofacial Surgeons.

DESCRIPTION

AllPrime Select Square Wrench (9780751) or Solid Wrench has a 3.9mm universal square fit and an anatomical handle, dimensioned for the torque required during use.

It does not have a torque graduated ruler and for this reason, it is indicated only for use with AllPrime Select Implant Remover. The Square Wrench is manufactured in stainless steel surgical.

APPLICATION

AllPrime Select Square Wrench is indicated for the activation of the AllPrime Select Implant Remover for removal / extraction of Dental Implants already installed.

CONTRAINDICATIONS

AllPrime Select Square Wrench is not suitable for torque application on other products. Its exclusive indication is for AllPrime Select Implant Remover.

ASSEMBLY AND DISASSEMBLY

The Square Wrench was developed and designed in a single piece, which provides greater resistance and safety to the user, guaranteeing torque greater than 150 N.cm, therefore, there are no parts composition.

HANDLING

Once sterilized, the Square Wrench should be handled only in a sterile environment, by properly trained professionals and in appropriate attire at the moment of removal / extraction of AllPrime Select dental implants already installed. All procedures must be performed by qualified professionals and the handling of the Square Wrench is inherent to the training of the professional, who will be responsible for using the instruments in the best possible way, respecting AllPrime Select instructions for each instrument. After fitting the Implant Remover on the 3.9mm square connection, support the Square Wrench with the finger of one hand, and with the other hand apply a circular, uniform and gradual load counterclockwise. The required torque will depend on the biological and physical conditions to which the implant was subjected, so there is no specified torque.

SANITATION AND STERILIZATION

The AllPrime Select Square Wrench is reusable, provided non-sterile and should be sanitized and sterilized before and after use by following the

steps mentioned in the link below:

Novo AllPrime Select's Surgical Kit

PRESENTATION

The Square Wrench is individually packaged in tyvek® surgical paper packaging and transparent film display. The instrument undergoes a decontamination process that eliminates any type of dirtiness, but the sterilization process, described above, must be followed as soon as it is first used.

PRECAUTIONS

- Do not use if the product packaging is broken;
- This product should be used sterile. Do not sterilize under dry heat (stove). Always use steam and pressure autoclave;
- Chemical sterilization is not recommended;
- Do not autoclave the product in its original packaging;
- Due to the reduced opening of the mouth in the posterior region, the use of the instrument may be difficult;
- Ensure that instruments are not swallowed or aspirated by the person receiving the product (patient);
- Before each procedure check the conditions of the AllPrime Select instruments, always respecting their useful life. Replace instruments in case of damage, missing markings, deformation and wear;
- Before each procedure, make sure that the Implant Remover is fully engaged and fully inserted in the Spanner Wrench, otherwise deformation or even breakage of the instrument may occur in the application of the torque;
- At the time of installation, be sure to align the Implant Remover to the long axis of the Implant, thus avoiding damaging the system;
- The Implant Remover must be perfectly inserted and inserted completely to the Square Spanner Wrench to apply the necessary torque;
- Always use the sequence of AllPrime Select products. The use of instrumentation and / or prosthetic components from other manufacturers does not guarantee the perfect function of the AllPrime Select implant system and disclaims any warranty of the product;
- Do not apply torque with the Square Spanner Wrench on other products, only in conjunction with the Implant Remover;
- Check the occlusal and mesio-distal spaces before use to avoid oral damage to the patient;
- Apply the counterclockwise torque, always in a soft but firm way until the osseointegration rupture and complete removal of the implant;

ADVERSE EFFECTS

Adverse effects will only occur if the choice of instruments is inadequate.

STORAGE CONDITIONS

The product should be stored in a dry place and in the original packaging at room temperature and not exposed to direct sunlight. Incorrect storage may influence the original characteristics of the packaging and the product.

DISPOSAL OF MATERIAL

The professional must always evaluate the conditions of the instruments after each use. If the instrument is detected to be damaged, markings erased, deformed or excessive wear, must be disposed of in accordance with current regulations, complying with the environmental requirements and the criterion of potentially hazardous products, taking into account the different levels of contamination, physical risks and selective collection.

LIFE CYCLE

The Square Wrench should be disposed when it loses its functionality.

SHELF LIFE

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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