IFU


(Instruction for use)

Seletor de Altura / Tunnel Check / Selector de Altura

PORTUGUÊS

ENGLISH

ESPAÑOL

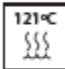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Favor contactar seu distribuidor autorizado.

This product has been designed to integrate clinical, mechanical and biological concepts, so, for best results it should only be used with original associated components according to AllPrime Select instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthodontists and Oral Maxillofacial Surgeons.

DESCRIPTION

The AllPrime Select tunnel checks are available in 2 models, one for fitting in Morse taper implants and the other for fitting in internal or external hexagon implants.

They have a cylindrical shape where, in the body of their upper portion, the product is marked by grooves and laser engravings that refer to the heights of prosthetic components straps in the AllPrime Select system, so that the user can plan with exactitude which component more attend to the functional and aesthetic aspect for continuity of the restoration procedure. Still at its upper tip, the tunnel checks have a hole dental floss installation for safety. At its lower end, the tunnel check has a geometry to mate with the implant's internal or analog fitting, without the need for a thread or any bolt, that is, the coupling takes place very quickly and easily. The tunnel checks are an integral part of the AllPrime Select prosthetic kit. The raw material used is stainless steel surgical series 300, according to ASTM F899 303.

The AllPrime Select tunnel checks are shown in the table below:

Sales Code	Connection	External Diameter
9780768	Morse Taper	3,0
9780767	IH and EH	4,0

APPLICATION

The AllPrime Select tunnel check is a surgical or laboratory instrument, indicated to be used to perform the selection of the gingival height of the prosthetic components that will be installed on the implant. Check the compatibility between the chosen prosthetic interfaces because the tunnel check is available for the Morse taper and internal or external hexagon prosthetic interfaces and are only compatible with the respective implants of the same lines. It can be used both in the mouth and in the laboratory for any type of procedure where it is necessary to select the prosthetic component to be installed.

CONTRAINDICATIONS

The tunnel check does not present contraindications, provided they are used correctly for the purposes indicated.

HANDLING

If used in the mouth, the tunnel check should be sterilized and handled only in a sterile environment, by properly trained professionals and in appropriate attire at the time of the procedure.

All procedures must be performed by qualified professionals and the handling of the product is inherent in the training of the professional who will be responsible for using the instruments in the best possible way, respecting the AllPrime Select indications for each instrument.

Before using the product, it is recommended to tie a

dental floss into the hole in the washer, to avoid the risk that the patient can swallow or suck the part.

After the installation of the implant and suture of the gingiva or preparation of the model in plaster, introduce the tunnel check, by its lower region of smaller diameter, in the internal connection of the implant or analogue and visually verify the height of the mucosa, thus selecting, the respective height of the components to be used.

SANITATION AND STERILIZATION

They are reusable, provided non-sterile and should be sanitized and sterilized before and after use by following the steps mentioned in the link below:

PRESENTATION

They are individually packaged in tyvek® surgical paper and transparent film display. The instrument undergoes a decontamination process that eliminates any type of dirtiness, but the sanitation process, described above, must be followed as soon as it's first used.

PRECAUTIONS

- Do not use if the product packaging is broken;
- This product should be used sterile. Do not sterilize under dry heat (stove). Always use steam and pressure autoclave;
- Chemical sterilization is not recommended;
- Do not autoclave the product in its original packaging;
- Before using the product, make sure it has the same implant prosthetic platform;
- The use of periapical radiography is recommended for the evaluation of the relationship between the mucosa, the contour of the bone border and the height selector.
- Due to the reduced opening of the mouth in the posterior region, the use of the instrument may be difficult;
- Ensure that instruments are not swallowed or aspirated by the person receiving the product (patient);
- Inadequate planning may compromise the performance of the implant / prosthesis assembly, resulting in system failure, such as loss or fracture of the implant, thread loosening or fracture of the prosthetic screw;
- Make sure you have all the necessary instruments for the surgery according to surgical planning;
- Before each procedure, check the conditions of the AllPrime Select instruments, always respecting their useful life. Replace the instruments in case of damage, erasure, deformation and excessive wear;
- Before each procedure, make sure that the parts fit together perfectly;
- The tunnel check does not have a fixation system, it is only inserted into the implant or analog fitting;
- Always use the sequence of AllPrime Select products. The use of instruments and / or components from other manufacturers and systems does not ensure the

perfect function and exempts AllPrime Select from offering any product warranty.

ADVERSE EFFECTS

Adverse effects will only occur if the described use recommendations are not adhered to.

STORAGE CONDITIONS

The product should be stored in a dry place and in the original packaging at room temperature and not expose to direct sunlight. Incorrect storage may influence the original characteristics of the packaging and the product.

DISPOSAL OF MATERIAL

The professional must always evaluate the conditions of the instruments after each use. If the instrument is detected to be damaged, markings erased, deformed or excessive wear, must be disposed of in accordance with current regulations, complying with the environmental requirements and the criterion of potentially hazardous products, taking into account the different levels of contamination, physical risks and selective collection.

LIFE CYCLE

The AllPrime Select tunnel check should be disposed of when they lose their functionality.

SHELF LIFE

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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