IFU

(Instruction for use)

Torquímetro Universal / Universal Torque Wrench / Torquímetro Universal

PORTUGUÊS

ENGLISH

ESPAÑOL

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Favor contactar seu distribuidor autorizado.

This product has been designed to integrate clinical, mechanical and biological concepts, so, for best results it should only be used with original associated components according to AllPrime Select instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthodontists and Oral Maxillofacial Surgeons.

DESCRIPTION

The AllPrime Select universal torque wrench (9780712) has in its upper part, a graduated ruler, that contains the marks 1, 2, 3, 4, 5, 6, 7 and 8, they correspond to the respective torques applied in the lateral elastic axis, with an increase of 10x, that is, the markings correspond to 10, 20, 30, 40, 50, 60, 70 and 80 N.cm. for visualization and torque control.

It also has an exclusive gear that serves both for coupling the 3.9 mm universal square fitting drivers and the contra-angle fitting drivers. This gear can be easily disassembled from the torque wrench and also serves as a bi-digital handle for all drivers. The torquimetro has an inverter button that allows the alterations of the directions of rotation of the ratchet (clockwise and counterclockwise). The inverter button has an arrow that indicates the direction of the torque when looking at forward the torque wrench. The torque wrench is an integral part of the AllPrime Select surgical kit and the AllPrime Select prosthetic kit.

The universal torque wrench is made of surgical stainless steel. The lateral elastic axis for torque application is made of nitinol, which is an alloy of Ni (nickel) and Ti (titanium) and presents the property of "shape memory effect", that is, however, it later regains its original shape.

APPLICATION

The AllPrime Select universal torque wrench is an instrument that helps in the insertion and torque control. It can be used in surgical and prosthetic procedures. It is compatible with instruments that have 3.9 mm universal square fitting or counter-angle fitting.

CONTRAINDICATIONS

The torque wrench does not present contraindications, provided they are used correctly for the purposes indicated.

ASSEMBLY AND DISASSEMBLY

For assembly:

- Insert the inner shaft and tighten the locking screw until it snaps into the torque wrench body.
- Pull the inverter button and engage the gear in the hole of the torque wrench head. Release the inverter button so that the gear engages the ratchet.

For disassembly:

- The torque wrench has a safety system that prevents the locking bolt from loosening with the use, and the ratchet disconnects from the gear, so, for disassembly, the user must remove first the gear.
- Unscrew the locking screw and remove the inner shaft.

HANDLING

Once sterilized, the universal torque wrench should be handled only in a sterile environment, by properly trained professionals and in appropriate attire at the time of surgery to install dental implants.

All procedures must be performed by qualified professionals and the handling of the product is inherent in the training of the professional who will be responsible for using the instruments in the best possible way, respecting the AllPrime Select indications for each instrument.

After fitting the driver into the torque wrench gear, support the gear with the finger of one hand, and with the other hand pull the side elastic shaft to monitor torque through the graduated shaft, with each circular movement of the gear. To reverse the direction of the ratchet rotation, pull and rotate the inverter button connected to the internal shaft by 180°, leaving the arrow facing the direction of the desired torque.

SANITATION AND STERILIZATION

The AllPrime Select universal torque wrench is reusable, provided non-sterile and should be sanitized and sterilized before and after use by following the steps mentioned in the link below:

[Surgical Kit Cleaning Manual](#)

PRESENTATION

The AllPrime Select universal torque wrench is individually packaged in tyvek® surgical paper and transparent film display. The instrument undergoes a decontamination process that eliminates any type of dirtiness, but the sanitation process, described above, must be followed as soon as it's first used.

PRECAUTIONS

- Do not use if the product packaging is broken;
- This product should be used sterile. Do not sterilize under dry heat (stove). Always use steam and pressure autoclave;
- Chemical sterilization is not recommended;
- Do not autoclave the product in its original packaging;
- Sanitize and autoclave the torque wrench always disassembled;
- Due to the reduced opening of the mouth in the posterior region, the use of the instrument may be difficult;
- Ensure that instruments are not swallowed or aspirated by the person receiving the product (patient);
- When the torque wrench is reversed, the gear loosens and may fall. Therefore, always perform this procedure out of the patient's mouth.
- Insertion torques higher than recommended or use of improper connection drivers can cause damage to the material and make the system unusable;
- Inadequate planning may compromise the performance of the implant / prosthesis assembly, resulting in system failure, such as loss or fracture of the implant or component, thread loosening or fracture of the prosthetic screw;
- Make sure you have all the necessary

instruments for the surgery according to surgical planning;

- Before each procedure, check the conditions of the AllPrime Select instruments, always respecting their useful life. Replace the instruments in case of damage, erasure, deformation and excessive wear;
- Before each procedure, make sure that the driver is fully engaged and tightened in the torque wrench, otherwise deformation or even breakage of the driver can occur;
- At the time of installation, be sure to align the key along the long axis of the component, thus avoiding damaging the system;
- The key must be fully seated and inserted into the implant or component to apply the required torque;
- The use of excessive force can compromise the surgical act;
- Always use the sequence of AllPrime Select products. The use of instruments and / or components from other manufacturers and systems does not ensure the perfect function and exempts AllPrime Select from offering any product warranty.

ADVERSE EFFECTS

Adverse effects will only occur if the choice of instruments is inadequate, which can result in damage to the implant or component to be installed.

STORAGE CONDITIONS

The product should be stored in a dry place and in the original packaging at room temperature and not exposed to direct sunlight. Incorrect storage may influence the original characteristics of the packaging and the product.

DISPOSAL OF MATERIAL

The professional must always evaluate the conditions of the instruments after each use. If the instrument is detected to be damaged, markings erased, deformed or excessive wear, must be disposed of in accordance with current regulations, complying with the environmental requirements and the criterion of potentially hazardous products, taking into account the different levels of contamination, physical risks and selective collection.

LIFE CYCLE

The universal torque wrench should be disposed of when they lose their functionality.

SHELF LIFE

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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