# Adesivo à base de álcool

DENTSCARE LTDA - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 – Joinville – SC - Authorization of Functioning MS P5X44XY0XX28 CNPJ/TAX ID: 05.106.945/0001-06 - BRAZILIAN INDUSTRY Registration at ANVISA nº 80172310074 Technical Authority: Friedrich Georg Mittelstädt CRQ: 13100147-SC - Brand: AIIPrime

0800 702 0227

www.aiiprimedental.com

Rev.: 00

BR

ES

EN

Irritante

Irritante

Irritant

BR

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**EN Instructions for Use**

**Light-cured Adhesive for Enamel and Dentin**

**Professional Use Only**

*Read all this information carefully before using the product. Keep it for later reference until total consumption of the product and/or until there*

**Product Description**

Adhesive A||Prime is a conventional two-step light curing adhesive responsible for promoting union between tooth structure (enamel and dentin) and restorative materials. The primer and the bonding are combined in a single flask.

Prior use of acid etchant is required both on tooth's enamel and dentin. It can be used for direct restorations in composite resins and light curing glass ionomer or for indirect procedures when associated with resin cement.

**Product Presentation**

Package containing a flask with 4mL of the product and instructions manual.

**Basic Composition**

Active Ingredients: MDP (10-Methacryloyloxydecyl dihydrogen phosphate, Methacrylic Monomers, Photoinitiators, Coinitiators and Stabilizers.

Inactive Ingredients: Inert Load (Silica Nanoparticles) and Vehicle (ethanol).

**Product's Indication**

A|| classes of direct restoration with composite (class I, II, III, IV and V); Adhesive luting (in addition with resin cement) of prosthetic pieces (post/cores, crowns, onlays/inlays, veneers, etc.) made of fiberglass, ceromer, ceramic, resin and metal; Adhesive repairs in ceramic and composites.

**Precautions and Contraindications**

1. Before using the bonding agent, make sure that the dentin surface is slightly shiny (moistened), however it cannot be wet;
2. The air used in the whole procedure must be oil and water free;
3. The recommended light curing time for this product depends on the power of the light emitted by the equipment used. The recommended power density of the light curing unit is above 400mW/cm<sup>2</sup> which guarantees the quality of the generated polymer and the quality of the bonding. The light curing time must be longer if the light curing unit presents lower power density. There is no possibility of excessive exposure. Light curing units must have its light intensity checked by a radiometer frequently. It is recommended that the patient and the

surgeon use protective goggles during the emission of blue light by the light curing unit;

4. Do not apply excess adhesive to avoid compromising the quality of adhesion and prevent any aesthetic and functional problems. Excessive application of air blasts should also be avoided to prevent bubbles in the adhesive;
5. Do not leave Adhesive A||Prime at the reach of patients and children;
6. Adhesive A||Prime contains methacrylate monomers. Avoid using the product in patients with allergy to acrylates / methacrylates. To reduce risks, minimize the exposure time of the dental tissues to the material when it is not polymerized;
7. Wear protective gloves when handling the product. If the product comes into accidental contact with eyes or other oral tissues, wash the area with water abundantly;
8. Make sure to close the bottle cap after use;
9. In deep cavities make the protection of the dentin-pulp complex with a base or liner material. Do not use material that contains Eugenol as a basis to avoid insufficient curing on the surface. The application of high concentration of H<sub>2</sub>O<sub>2</sub> should also be avoided because it can interfere with the curing process of the adhesive;
10. No need to shake the bottle before use.

**Side Effects**

Adhesive A||Prime can cause sensitization reactions in patients sensitive to any components of the formula. In this case, the material should not be used.

**Instructions for Use**

Before starting the treatment carefully read the Warnings, Precautions, Contraindications and possible Side Effects.

A. Light curing direct restorations in enamel and/or dentin:

1. Make prophylaxis of the teeth with pumice and water or with sodium bicarbonate jet;
2. Prior to registering the teeth color make sure teeth are clean and hydrated;
3. Anesthesia can be administered if required;
4. Properly isolate the area: direct bonding restorations can be adequately performed with rubber dam or relative isolation depending

on the case. There are cases where the use of rubber dam is essential;

5. Tooth Preparation: Prepare the cavity with minimal reduction of tooth structure as possible;
  6. Adhesive Application: See specific procedure in item 1 of this manual.
- B. Instructions for Bonding of Indirect Restorations:

1. Tooth Preparation: Make the isolation of the tooth with rubber dam or relative isolation with lip retractor and use a gingival retraction cord when the margin of the preparation is subgingival, to ensure no contamination while tooth preparation;
2. Properly clean the preparations with pumice and prophylaxis brush/cup;
3. The treatment of the internal surface of the prosthetic parts should follow recommendations according to the classification of the material. Follow the manufacturer's recommendations on the cementation process;
4. Adhesive Application: See specific procedure in item 1 of this manual.

C. Instructions for Bonding of Prefabricated Posts:

1. Perform a radiographic examination (periapical radiographs) and check the quality of the endodontic treatment;
2. Isolate the operative field. Prioritize the use of rubber dam;
3. Partially remove the gutta percha from the conduit using a specific drill or appropriate instrument maintaining 3 to 5 mm of gutta percha in the apical region, ensuring the sealing of endodontic fillings;
4. Select the prefabricated post according to the diameter of the conduit;
5. Prepare the root canal with appropriate drill to adapt it to the post shape. Avoid unnecessary removal of tooth structure.
6. Adhesive Application: See specific procedure in item 1 of this manual.

1. Adhesive Application

1.1 ACID ETCHING:

a. Direct and Indirect Restorations: Apply phosphoric acid or 15 seconds in both enamel and dentin, according to the type of preparation. Wash the surface with water and dry the cavity so that the dentin does not get dehydrated but also there is no accumulation of water on the surface. If there is need to re-wet the dentin for the application of the adhesive system, use cotton pellets moistened with distilled water or Chlorhexidine 2%. By rewetting the dentin we also re-wet the enamel, which can be

treated using the moist technique.

b. Cementation of prefabricated post: Apply phosphoric acid gel into the root canal and remaining tooth structure for 15 seconds. Wash the area carefully and remove the excess water with absorbent paper cones.

1.2 ADHESIVE APPLICATION:

a. Direct and Indirect Restorations: Dispense Adhesive A||Prime in a Dappen pot or directly onto a disposable microapplicator. Apply two layers - one drop for each - of adhesive on the slightly moistened tooth surfaces. The first layer should be applied vigorously by rubbing the adhesive microapplicator saturated with the product for 10 seconds. Next, the second layer of adhesive - with a new drop - is applied for another 10 seconds and then the area is air blasted gently for 10 seconds to evaporate the solvent. Light cure Adhesive A||Prime with blue light for 10 seconds.

b. Cementation of prefabricated posts: With the aid of a zddisposable microapplicator, apply two layers - one drop for each - of the product into the root canal vigorously rubbing the adhesive on the walls of the conduit for 10 seconds. Evaporate the solvent with gentle blasts of oil-free air for 10 seconds, and then light cure it for 20 seconds.

OBS.: During the adhesive application for post cementation, avoid accumulation of the product in the apical region. Excess of adhesive in this region difficult the evaporation of the solvent and compromises its polymerization; The recommended cure time for this product depends on the power of the light emitted by the equipment used. The recommended power density of the light curing unit is above 400mW/cm<sup>2</sup> which guarantees the quality of the polymer generated and also the quality of the bonding.

1.3 BONDING OF RESTORATIVE MATERIALS:

Perform restoration or cementation according to specific instructions of the product selected for the procedure (composite resins and/or cements).

**Conservation and Storage**

Adhesive A||Prime must be stored in temperatures between 15° to 30°C / 59°-86°F; Protect product from external contamination;

The flask containing the product must be closed carefully after use in order to prevent solvent evaporation; Do not expose the product to intense light or high temperatures; Do not use product after expiry date; For cleaning, Adhesive A||Prime can be removed with alcohol; Adhesive A||Prime expires after 3 years from date of manufacture which is described on the box of the product.

**Warnings**

Do not use if product is expired. In order to dispose of product properly follow specific country guidelines. Do not reuse empty packages. Keep out of the reach of children. *This material was manufactured for dental use only and should be handled in accordance with the instructions. The manufacturer is not liable for damage caused by other uses or by mishandling. In addition, the User is obliged to prove, if this material is compatible with the desired use before employment it and under his responsibility, especially when such use is not indicated in this instruction. Overview of data does not constitute any kind of guarantee and, therefore, have no binding.*