Resina microhibrida

DENTSCARE LTDA - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 – Joinville – SC - Authorization of Functioning MS P5X44XY0XX28
 CNPJ/TAX ID: 05.106.945/0001-06 - BRAZILIAN INDUSTRY
 Registration at ANVISA nº 80172310073
 Technical Authority: Friedrich Georg Mittelstädt
 CRQ: 13100147-SC - Brand: AiPrime

Rev.: 00

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EN Instructions Manual

Light-cured Composite for Anterior and Posterior Teeth For Professional Use Only

Read carefully all the information in this product information sheet before using the product. Keep it for reference until total consumption of the product or until there is no more interaction between the product and your last patient.

1. PRODUCT DESCRIPTION

AllPrime Resin is a nanohybrid, radiopaque composite for anterior and posterior restorations. Its composition was based in modern concepts so that methacrylate monomers, silanes and fillers with adequate size distribution were combined to produce a product with differential physical, mechanical and optical properties. AllPrime Resin presents a simplified system of shades, offering enamel, dentin and incisal shades. The inorganic filler loading of AllPrime Resin is 77.5 to 78.5% by weight or 56 to 59% by volume and presents particles sizes between 40 nm and 3,0 microns with an average particle size of 0.8 µm.

2. COMPOSITION

- Active Ingredients: Bis-GMA monomer (Bis-phenol A di-Glycidyl Methacrylate), Bis EMA (Bis-phenol A di-Glycidyl Methacrylate ethoxylated), TEGDMA (Triethylene glycol dimethacrylate), camphorquinone, co-initiators and silane.

- Inactive Ingredients: Barium-aluminum silicate glass and silicon dioxide nanoparticles.

3. INDICATIONS

It is a product for professional dental use, indicated for teeth in need of functional and esthetical restoration. The main indications of the product are:

-Occlusal, proximal and occluso-proximal restorations of medium and small sizes;

- Anterior Class III, IV and V restorations;

- Direct composite veneers;

- Re-attachment of tooth fragments;

- Reducing and/or closing diastemata;

- Correcting and/or altering the form of one or several teeth;

-Structural defects: Amelogenesis Imperfecta, enamel hypoplasia, non-carious cervical lesions;

- Correcting and/or altering the proportional height and/or width of teeth;

- Restoration of primary teeth;

- Correcting and/or altering the shade of part or the whole tooth surface.

4. PRECAUTIONS AND WARNINGS

-In case of allergic reactions, discontinue the use of the product.

-Avoid the use of eugenol containing materials, such as liners or temporary restoratives, as they hinder the light curing of the product.

- Avoid the contact of uncured AllPrime Resin with the skin, mucosa and eyes. Uncured resins may cause a slightly irritating effect and promote sensitivity due to the presence of methacrylates.

- Keep out of reach of children.

5. ALLPRIME RESIN SHADE SYSTEM

The enamel and dentin shades follow the VITA shade guide. Additionally, AllPrime Resin composite presents the INCISAL shade to reproduce dental areas of high translucency as the incisal third.

Table 1: Color availability, translucency, quantity and exposure time.

Category	Colors	Translucency (%)	Quantity	*Exposure Time
Dentin	DA 1	43-46	4g	40 s
	DA 2			40 s
	DA 3			40 s
	DA 3,5			40 s
	DB 2			40 s
Esmamel	EA 1	53-56	4g	20 s
	EA 2			20 s
	EA 3			20 s
	EA 3,5			20 s
	EA 4			20 s
	EB 1			20 s
	EB 2			20 s
	EB 3			20 s
	EC 2			40 s
	EC 3			40 s
Effect	Incisal	66	4g	20s

* For thicknesses not beyond 2,0 mm depth and light cured with a light intensity of 500 mW/cm².

The exposure time described in Table 1 must be followed to guarantee an adequate curing depth, degree of conversion, color stability and mechanical properties of the composite resin. Overexposure is not possible.

6. PRESENTATION

Enamel Refil: EA1, EA2, EA3, EA3.5, EA4, EB1, EB2, EC2 and Incisal;

Dentin Refil: DA1, DA2, DA3, DA3.5, DB2;

7. INSTRUCTIONS FOR USE

Read carefully the Warnings, Precautions, Contra-Indications and Possible Side Effects written in this Product information sheet, before the beginning of the treatment.

SHADE SELECTION:

-Before shade selection, perform a teeth prophylaxis. Teeth should be clean, hydrated and not isolated since dehydration may alter their shade.

-The shade of the tooth can be identified by means of VITA shade guide;

-Alternatively, the shade selection can be done using the own product: a

small portion of composite resin can be placed over the buccal surface of the tooth to be restored (without acid conditioning and adhesive application). After light curing for 30 seconds, wet the composite resin with the patient's saliva and observe if the shade selected match with that of the tooth structure. If not, repeat the procedure using another composite resin shade.

PRODUCT APPLICATION:

1 -Isolate the operative field. The isolation must guarantee a dry surface area which is clean and visible during the whole treatment. If necessary, perform rubber dam isolation;

2 -Use conventional cavity preparation for direct adhesive restorations;

3 - Apply 37% acid conditioner initially in the enamel margins and then apply inside the cavity.Wait for 15 seconds.Wash with water abundantly removing all acid from the tooth structure.Dry the dental structure leaving the dentin slightly moist;

4 - Apply the adhesive system according to the manufacturer's instructions;

5 - Place and light-cure the composite resin:

- We recommend the incremental placement technique as it permits a more uniform curing of the resin increment. Follow the exposure times described in Table 1 for resin increments not beyond 2 mm depth.

- Restore the cavity according to the selected shade applying small layers of AllPrime Resin (maximum 2 mm thickness) adapting them in the cavity carefully.

- For finishing and polishing procedures, sand disks and felt disks with polishing pastes can be used.

Note: Checking occlusion is an extremely important procedure. Restorations in supra-occlusion cause excessive stress in the tooth and the restoration which in turn might cause pain. Do not leave them in infra-occlusion either.

8. STORAGE AND MAINTENANCE

Keep the product in a fresh place. The syringe should be always tightly capped and protected. Keep out of direct sunlight. Store the product at temperature between 5 and 30°C / 41-86°F. Do not freeze the product.

9. EXPIRATION DATE

3 years from the production data.

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the instructions for Use.

Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. Besides that, the user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.